

# Quick Start Guide

Please note this guide is supplementary and not a replacement to the IFU.  
Always review the IFU before utilizing Pulpdent products in your practice.



1. Prepare cavity.



2. Matrix placement for Class II restoration.. Establish sufficient separation and burish matrix for optimal contacts.




3. Apply adhesive system of choice. Follow manufacturer instructions.




4. Bulk fill with ACTIVA™ BioACTIVE Bulk Flow™.



5. Allow 15 seconds to self-level. Light cure for 10 seconds.

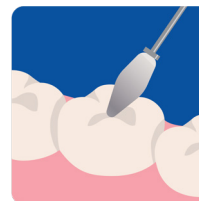

6. Sculpt final anatomy. *Optional:* place a 1-2mm capping layer of universal composite if desired.

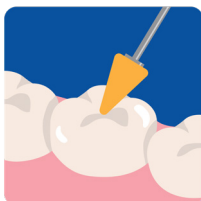

7. Polish completed restoration.



LEARN MORE

Scan the QR code for more information on ACTIVA BioACTIVE Bulk Flow

