Calcium Hydroxide Canal Treatment Paste

TempCanal Enhanced has a pH>12 and provides the proven benefits of calcium hydroxide for successful root canal therapy. Use routinely as an intracanal dressing and for treatment of periapical lesions and abscesses, internal and external root resorption, traumatic injuries, root fractures and perforations, apexification, and weeping canals.

The paste flows through 27-gauge x 1” (0.4 x 25mm), 2-side-vent, endodontic irrigation needles for direct, one-handed delivery into the root canal. Closed-end needles prevent dispensing beyond the apex. Non-drying formula will not clog 27-gauge needles.

XP-TE-IN-09w
REV: 03/2026

PRODUCT DESCRIPTION

TempCanal Enhanced is a calcium hydroxide-based paste used by dental professionals in root canal procedures and as pulpal dressing for all pulp capping, pulpotomy and pulpal curettage procedures.

INDICATIONS

For routine use between office visits to cleanse canals and for use in complicated cases when extended calcium hydroxide treatment is indicated.

CONTRAINDICATIONS

Not for use as a permanent, hard-setting root canal filling material.
For any potential allergies, refer to associated MSDS.

INSTRUCTIONS FOR USE

1. Prepare and cleanse canal in the usual manner.
2. Attach luer lock dispenser needle securely onto the syringe. (Alternatively, paste can be placed in the canal using a lentulo paste filler.)
3. Insert needle into the canal to the desired depth. The needle is closed end with two side vents.
4. Depress syringe plunger to dispense paste.
5. Fill canal while slowly withdrawing the needle.
6. Seal access opening with your preferred cement.
7. Discard needle and recap syringe immediately after use
8. TempCanal Enhanced is easily removed with file and irrigation.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
 - Shelf life of unopened product: 2 years from date of manufacture.
 - Re-cap immediately after use.
- Note: Use a new applicator tip for each patient.


CAUTION

May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:
Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com
Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multi-ple-use-dental-dispenser-devices>
Note: Refer to the Safety Data Sheet for a list of materials present in this product.

CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

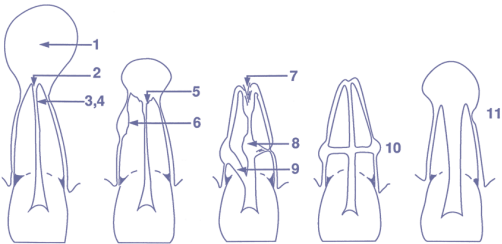1. exudation control
2. large periapical lesions
3. antibacterial dressing
4. temporary root filling
5. apical inflammatory resorption
6. inflammatory resorption following trauma
7. apical internal resorption
8. internal-external root resorption
9. perforations
10. transverse root fractures
11. incompletely developed pulpless teeth

(Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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PULPDENT® Corporation